


VASCULAR ACCESS
Accessories for implantable ports

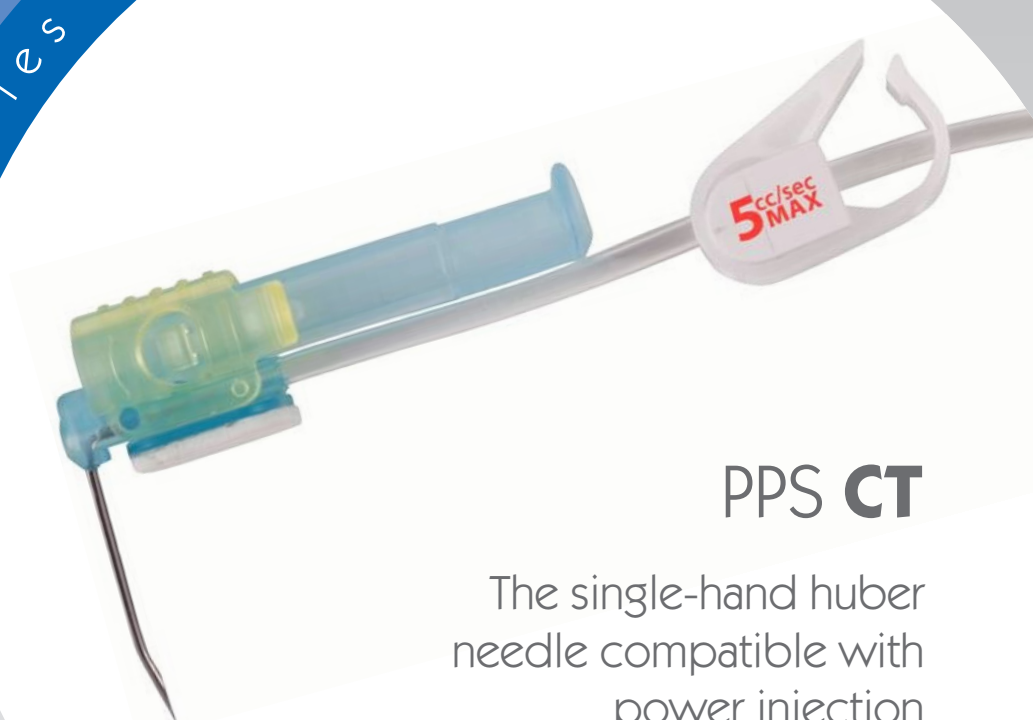

PPS **CT**

The single-hand huber
needle compatible with
power injection



Value Life

Safety Huber needle PPS CT



Possible positive pressure

- According to implantable port using recommendations⁽²⁾
- Withdrawal of the needle with only one hand

Prevention of needlestick injury

- Decreases blood exposure risk versus double and activated device
- No rebound effect when withdrawing the needle
- No risk of contact with the needle body
- Total protection until discarded in sharps container⁽³⁾

Withdrawal of the needle


Normal position for use (horizontal piston) before withdrawal.

Preparation: lift the piston from the horizontal position to the vertical position.

Using your thumb, lower the piston to put it in contact with the skin and simultaneously lift the extractor (with an upwards movement) until there is a CLIC corresponding to the total locking of the needle. This final operation eliminates any risk of accidental needlestick injury.

Positive pressure

Gauge	Ext. Ø	Length	Max. Flow rate	Product codes		Packaging
				Without Y-site	With Y-site	
22 G	0.7 mm	15 mm	2 ml/sec 300 PSI	VPE801507*	VPE811507*	12 / carton box
		17 mm		VPE801707	VPE811707	12 / carton box
		20 mm		VPE802007	VPE812007	12 / carton box
		25 mm		VPE802507	VPE812507	12 / carton box
		30 mm		VPE803007	VPE813007	12 / carton box
		35 mm		VPE803507	VPE813507	12 / carton box
20 G	0.9 mm	15 mm	5 ml/sec 300 PSI	VPE801509*	VPE811509*	12 / carton box
		17 mm		VPE801709	VPE811709	12 / carton box
		20 mm		VPE802009	VPE812009	12 / carton box
		25 mm		VPE802509	VPE812509	12 / carton box
		30 mm		VPE803009	VPE813009	12 / carton box
		35 mm		VPE803509	VPE813509	12 / carton box
19 G	1.1 mm	15 mm	5 ml/sec 300 PSI	VPE801511*	VPE811511*	12 / carton box
		17 mm		VPE801711	VPE811711	12 / carton box
		20 mm		VPE802011	VPE812011	12 / carton box
		25 mm		VPE802511	VPE812511	12 / carton box
		30 mm		VPE803011	VPE813011	12 / carton box
		35 mm		VPE803511	VPE813511	12 / carton box

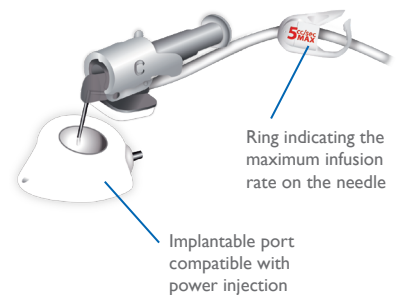

* Restricted to paediatric use

(1) See Instruction for use.
 (2) According to HAS 2000 (French Health Authority) and SF2H 2012 (French Society for Hospital Hygiene) guidelines.
 (3) Biomech study n° 148381 of 28 June 2012, p.64-66.

For further information, please contact: marketingbenelux@vygon.com

The specifications shown in this leaflet are for information only and are not, under any circumstances, of a contractual nature.

SA Vygon NV • Haachtsesteenweg 1650 Chaussée de Haecht • 1130 BRUXELLES • BELGIQUE
 Tel: +32 (0)2 706 09 50 • www.vygon.be

Vygon Nederland B.V. • Kerkhofstraat 21 • 5554 HG VALKENSWAARD • NEDERLAND
 Tel: +31 (0)40 208 93 80 • www.vygon.nl
 VY17048 • [in](https://www.linkedin.com/company/vygon) Vygon Benelux

