



MID & LONG TERM VASCULAR ACCESS
Securement Device for Drains



Value Life

securAcath[®] provides improved drainage catheter securement

Reliable securement of drainage catheters is a serious clinical problem. Sutures and adhesives have been used for years with moderate success. Catheter migration and dislodgement are frequent complications of drainage catheters.

How does the securAcath[®] work?


- Small, blunt, nitinol securement feet are placed just beneath skin right at the catheter insertion site
- Cover is snapped onto base to affix to catheter shaft
- No sutures or additional skin punctures are needed
- No adhesives needed for securement
- Remains in place for life of catheter
- Works with nephrostomy, general/abscess, biliary, pleural and other drainage catheters



FOLD



INSERT

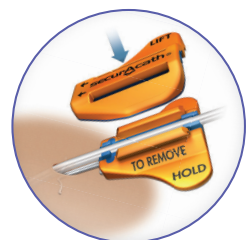

SNAP

● Benefits on Drainage Catheters

Decreased Catheter Dislodgments and Replacement Costs

- Lower cost of patient care
- Minimize therapy interruption

Increased Stability

- securAcath® secures right at the catheter/skin junction
- Minimizes catheter movement, pistoning and migration

Eliminates Suture Needle Sticks

- No sharps related injuries

Improved Efficiency



- securAcath® is much faster to use than sutures

360 Degree Site Cleaning While Secured

- Excellent cleaning access around the entire insertion site
- Catheter remains stable and secure during cleaning

Decreased Skin Irritation

- Sutures can become infected, cause skin irritation and breakdown
- Minimize use of skin-damaging adhesives

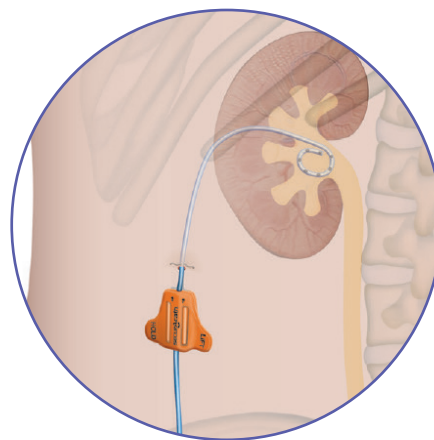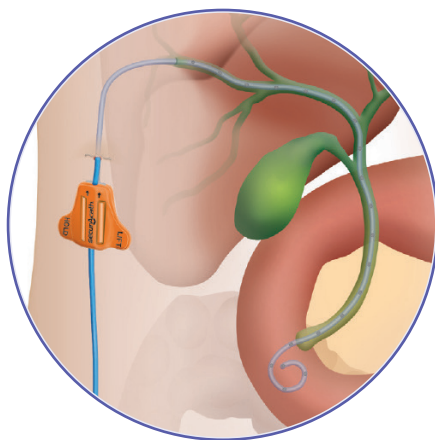

References

Code	Size	Quantity/box
400150	6F	10
400120	7F	10
400160	8F	10
400180	10F	10
400200	12F	10

Additional securAcath[®] product information

- Not made with natural latex rubber
- MRI Conditional

INTRAVASCULAR THERAPIES

For further information, please contact: marketingbenelux@vygon.com

The specifications shown in this leaflet are for information only and are not, under any circumstances, of a contractual nature.

SA Vygon NV • Haachtsesteenweg 1650 Chaussée de Haecht • 1130 BRUSSELS • BELGIUM

Tel: +32 (0)2 706 09 50 • www.vygon.be

Vygon Nederland B.V. • Kerkhofstraat 21 • 5554 HG VALKENSWAARD • NEDERLAND

Tel: +31 (0)40 208 93 80 • www.vygon.nl

VY21055 •  Vygon Benelux

